

Enhansit®

INSTRUCTION MANUAL

It's as easy as,

1.)



2.)



3.)



526 West Main Street

Plymouth, PA 18651

Toll Free: 1-888-217-1211

FAX: 570-779-7736

Email: sales@enhansit.com

www.enhansit.com


NOTICE

You have just purchased the most revolutionary lifting system that works with your existing recliner. As stated in our advertisement, if you own or plan to purchase a Rocker recliner, Wall Saver recliner, or standard recliner, a simple procedure of attaching the ENHAN-SIT® to the existing base of your chair is all that is required to make that chair function as a lift chair.

NOTE: THE ENHAN-SIT® IS NOT FOR USE ON SWIVEL RECLINERS.

We know you are very excited about using your new ENHAN-SIT® system, so we have made this simple systematic picture list to get the job accomplished easily. We encourage you to get familiar with the ENHAN-SIT® and the few components with it, and the undercarriage or base of your recliner or the base of one you intend to purchase before beginning installation.

Figure 1-1

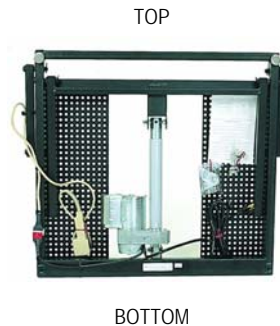

INSTALLATION



Simply tip the chair and lay on its back. Take a minute to look at the exposed base of your existing chair.

Reference: Figure 1-1 for the location of the TOP and BOTTOM of the ENHAN-SIT®. Carefully, place both hands around the front cross bar and position evenly the top/front of the ENHAN-SIT® against the bottom of your chair. It is important to place the perforated plate of the ENHAN-SIT® flat against and uniformly spaced to the base of your chair.



The ENHAN-SIT® comes equipped with U-bolts. While using your leg to hold the ENHAN-SIT® firmly against the base of the chair, attach the first 2 each U-bolts over the base of your chair and through the holes in the perforated plate of the front of the ENHAN-SIT®. Attach the 1/4-20 nuts to the U-bolts to secure the base of the chair to the perforated plate. Repeat this on the other side of the front of the chair. The U-bolt nuts should be finger-tightened only to aid in the alignment and attaching of the ENHAN-SIT® back/bottom.



The ENHAN-SIT® should now remain in place without the use of your leg to hold it. Position the U-bolts around the back/bottom of the base of the chair and through the holes in the perforated plate of the ENHAN-SIT®. Attach the 1/4-20 nuts to the U-bolts to secure the base of the chair to the perforated plate. Repeat this on the other side of the back of the chair. With the 4 each U-bolts in place, tighten all the nuts firmly and securely.

IMPORTANT
For the ENHAN-SIT® system to operate properly and safely, it must be attached evenly (side to side and front to back), and all 4 U-bolts must be attached and securely tightened around the base of your chair. DO NOT over tighten the nuts this could result in stripping or breaking of the U-bolts.



Carefully, remove the tie wraps from the power adapter, hand controls and power cord. The power cord attached to the ENHAN-SIT[®] will exit the back of the chair and plug into the power adapter. The power adapter will plug into your 120VAC power outlet. A 5-position quick disconnect connector is securely fastened to the back outside of the ENHAN-SIT[®]. Attach the hand controls to the quick disconnect connector and route the cord up the side of the chair. Use the cord clamp and Phillips wood screw (included) to fasten the cord underneath the chair arm.

If for any reason you must remove a motor or hand control, be sure to reattach it exactly as it was before removing. **FAILURE TO DO SO COULD RESULT IN A SERIOUS INJURY.** If you have any questions concerning the attachment of the ENHAN-SIT[®], contact your authorized dealer or the place of purchase.


OPTIONAL INSTALLATION

ENHAN-SIT[®] can also be lag bolted to a wood base; Reference Figure 2-1, below.

1.) Remove the chair legs, if any. 2.) Mark an appropriate fastening place on the base with pencil. 3.) Pre-drill properly sized holes in the base. 4.) Fasten lag screws (4 each included) through the perforated plate and into the pre-drilled holes in the base. 5.) Follow the previous instructions for hand control installation.

PLEASE NOTE:

The 4 each 1/4 inch lag bolts are designed to be used with recliners that have no built in reclining mechanism this type of recliner has an open or empty area under the chair. Always pre-drill a pilot hole before using the lag bolts!

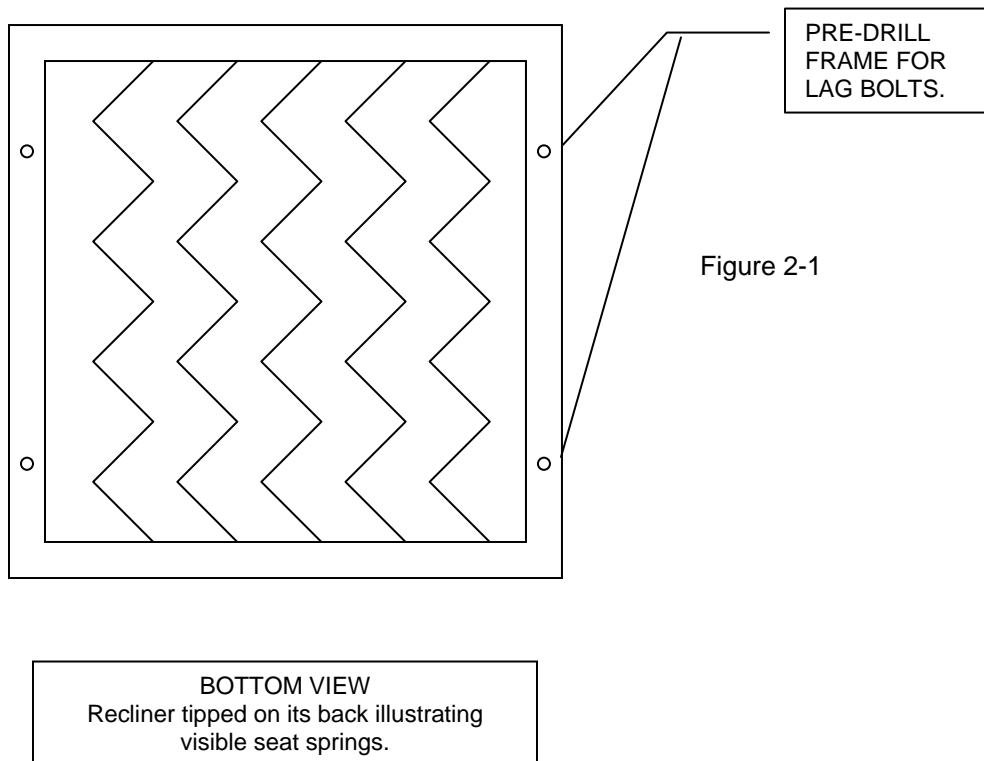

ATTENTION

The ENHAN-SIT[®] is designed and engineered by individuals who have many years of experience with such products. The Buzzz Corp.[®] understands clearly the need for such a product and how to easily and safely apply the latest technologies to bring you the patented ENHAN-SIT[®]. It is the sole responsibility of the consumer and user of the product to use this product in a safe and responsible manner. Please be certain to adhere to the following:

- 1) Follow all instructions carefully.
- 2) Any questions should be immediately directed to an authorized dealer. Alternatively, reference our website for Frequently Asked Questions (FAQ). www.enhansit.com
- 3) When raising or lowering your chair with the ENHAN-SIT[®], make certain that no objects – PETS or CHILDREN are UNDERNEATH or NEAR THE CHAIR!
- 4) NEVER LET CHILDREN PLAY WITH OR OPERATE THE ENHAN-SIT[®], IT IS NOT A TOY!
- 5) Always disconnect power source before servicing.
- 6) Any alterations to the product will void the warranty, and could cause serious injury.

We at The Buzzz Corp.[®] hope you like your new ENHAN-SIT[®]. If used properly, we know it will provide you with many years of trouble free use.

LIMITED WARRANTY

The Buzzz Corp.[®] (herein referred to as Buzzz) warrants to the original consumer purchaser that the ENHAN-SIT[®] (herein referred to as product) will be free from defects under normal use and service, for a period of (1) one year from date of purchase. The items covered by this warranty are the frame, motor and switch.

- Buzzz will repair or replace defective parts without charge at our discretion, as long as the original purchaser follows the warranty service procedure.
- The warranty only applies if the product was purchased as new equipment from an authorized dealer. Warranty does not apply to the purchase of used equipment.
- The warranty does not cover any damage or defect of any kind to the chair that attaches to the product. Proper care and adherence to the installation instructions must be practiced to insure proper and safe mechanical and electrical operation of this product.
- A warranty card (included) must be on file at the Buzzz.
- AFTER THE ONE YEAR WARRANTY PERIOD. BUZZZ SHALL NOT HAVE ANY OBLIGATIONS OR LIABILITIES UNDER ANY WARRANTIES, EXPRESSED.

WARRANTY INFORMATION

Please return the information contain on this card immediately to The Buzzz Corp.

This will allow us to handle your service needs quickly and easily.

The Buzzz Corp. reserves the right to refuse service if signed original warranty card is not on file.

Name: _____

Address: _____

City: _____ State: _____ Zip Code: _____

Serial No: _____

Purchased From: _____

Date of Purchase: _____

Dealer's Name: _____

I learned about the ENHAN-SIT[®] from:

City/State: _____

Television Doctor Other

I chose the ENHAN-SIT[®] because:

Friend Catalog Radio

Furniture Magazine Therapist

Internet Web Search

I have read the owner's manual and fully understand the instructions and warnings contained within.

Comments:

Signed: _____

Date: _____

PLACE
STAMP
HERE

THE BUZZZ CORP
526 WEST MAIN ST
PLYMOUTH, PA 18651